
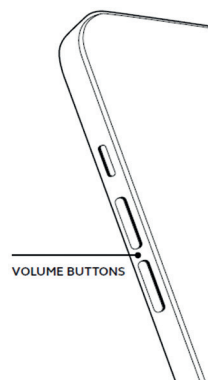


How to enjoy Hands-Free Calls for iPhone and iPad with Danalogic Actio

Please ensure your device is compatible by checking our website here go.gn.com/app-support

Function	How to
Answer a call	Use a short press on your hearing aid button or slide to answer on the iPhone/iPad
End a call	Press your hearing aid button to end the call. Alternatively, hang up by tapping the red button on your screen
Second incoming calls	If you are on a call and you have a second call incoming, you can decline it by using the long press on your hearing aid button or tap 'Decline' on the iPhone or iPad screen
Change the volume	Use the up and down volume buttons on the iPhone or iPad
Turn off hands-free calls in the iOS hearing aid settings	Triple click the iPhone push button to access the iOS hearing aid settings (you can also get access via the Control Centre). Select 'Input Options' then 'Off'.

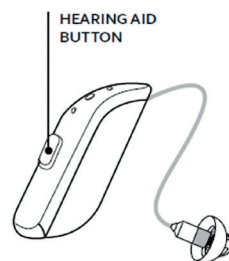

For further support, scan the QR code or visit our patient support portal at: danalogic.co.uk/patient-portal