

Get connected quick guide


How to connect your Danalogic hearing aids to the BeMore app.

INTRODUCTION

For information on which smartphones and mobile devices are compatible with which app, visit: <http://go.gn.com/app-support>
Before you can start using the app, you must pair your smart devices with your hearing aids. This is easily done – just follow the steps below.

HOW TO PAIR YOUR HEARING AIDS WITH YOUR iOS DEVICE

1. Turn your hearing aids off.
2. Turn on **Bluetooth®** by going to the **Settings** menu on your iOS device. The slider turns green when you activate Bluetooth.
3. Go back to the main **Settings** menu and then, tap on **Accessibility**.
4. Scroll down to **Hearing Devices**.
5. Turn your hearing aids on. Now, your iOS device will be able to detect your hearing aids.
6. Tap on **Pair**.
If you have two hearing aids, you will need to accept twice.
7. Go to the App Store and search for BeMore- for your hearing aids
8. Download the BeMore app, open it and tap **Get Started**.
9. Follow the instructions on the screen to connect to the app.
You are now ready to go.

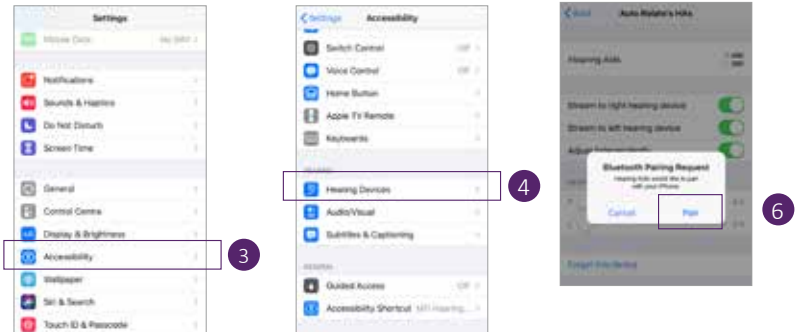

NOTE: These are the steps and how it looks in iOS 13 and newer



HOW TO PAIR YOUR HEARING AIDS WITH YOUR ANDROID™ DEVICE

1. Go to the Google Play Store, search for BeMore and download the app.
2. Once downloaded, open the app and tap **Get Started**.
3. Follow the instructions on the screen to connect. You are now ready to go.

NOTE: If you have previously paired any hearing aids, you need to unpair them in your smartphone's Bluetooth menu before you pair them again.

IF YOU NEED TO RE-PAIR YOUR iOS DEVICE

1. Start by removing the old pairing by going to the **Settings** menu.
2. Scroll down and select **Accessibility**.
3. Select **Hearing Devices**.
4. Tap on your hearing aids, scroll down and tap on **Forget this device**.
5. Turn your iOS device off and on again and re-pair the hearing aids to the device. If you need more information, contact your hearing care professional.