


Changing Domes for Receivers

Standard domes

Follow these steps to mount domes:


1. Push the new dome over the ridge on the receiver
2. Make sure that the new dome is properly and securely mounted

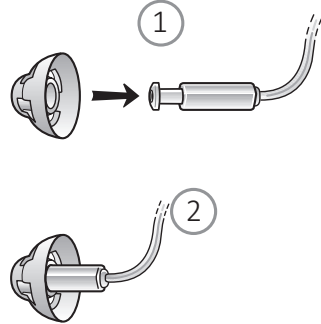


Tulip domes

The tulip domes are mounted in a similar manner to the standard domes, but a few extra steps are required. The tulip domes consist of two “petals”. Follow these steps to mount tulip domes:


Bend the largest petal forwards to open the tulip dome.

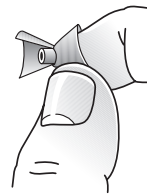


Push the fitting tube over the ridge.

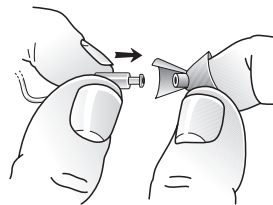

Check that the ridge are completely covered by the fitting tube.

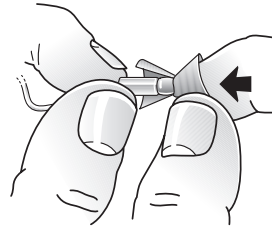

Then move the largest petal to its original position.

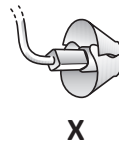

Wrong!



Correct!

Audiologist contact details: