

Earmould insertion and removal

To insert the earmould, hold it between your thumb and index finger and position the sound outlet into your ear canal.



Slide the earmould all the way into your ear with a gentle twisting movement. Ensure the top part of the earmould tucks in behind the fold of skin above your ear canal.




Once the earmould is in place, push the hearing aid forward and then slide it back to sit comfortably behind your ear.

Some people find using a mirror helps with insertion of an earmould and once correctly in place it should fit snugly but comfortably.

Earmould insertion and removal (continued)

To remove the earmould, lift the hearing aid from behind your ear and let it hang down beside your ear.

Using your thumb and index finger gently, pull the earmould loose from your ear. You may need to twist the earmould slightly to fully remove it.

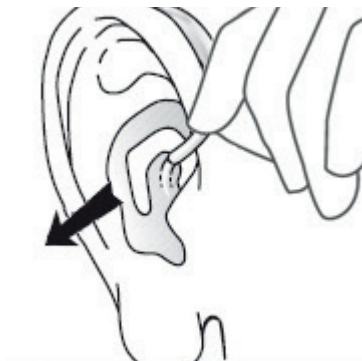

For further support, scan the QR code or visit our patient support portal at: danalogic.co.uk/patient-portal