



Cleaning and replacing slim tubes for danalogic Extend

Cleaning


Firstly, you can wipe down the slim tube and dome with a damp cloth. If the slim tube is clogged with wax or other debris, remove it from the hearing aid by carefully pulling the entire tube off of the hearing aid.

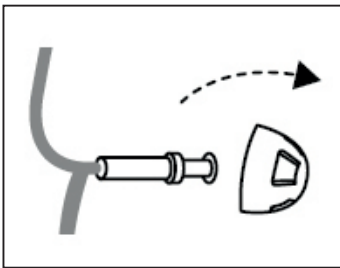

1. Next, remove the dome by pulling it off the ribbed end.

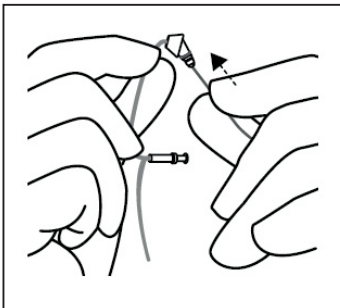


2. Now, push the cleaning wire through the top of the slim tube where it connects to the hearing aid. Thread the cleaning wire completely through to fully remove any debris.

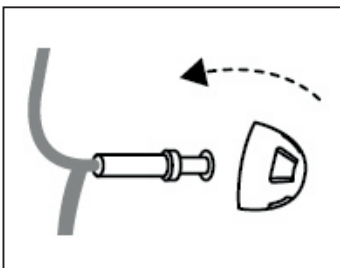


3. Now, push the dome back onto the slim tube.

Cleaning and replacing slim tubes for danalogic Extend (continued)

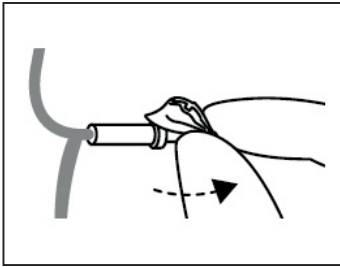

4. Check that the dome is secure by carefully lifting the lower part of the dome, and verifying that the collar completely covers the ribbed end of the slim tube.

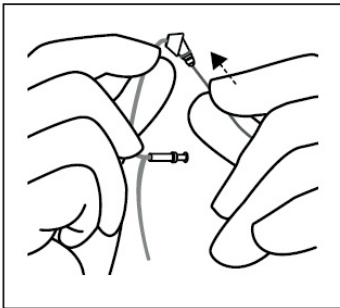

5. Finally, push the slim tube back onto the hearing aid until it clicks firmly in place.

Replacing

If you need to replace your slim tube pull the entire tube off of your hearing aids and replace with a new one, ensuring this clicks firmly into place. Follow the instructions above to ensure a new dome is correctly fitted onto your new slim tube.



For further support, scan the QR code or visit our patient support portal at: danalogic.co.uk/patient-portal