



How to position the telephone


If your hearing aids are fitted with a thin tube open dome or thin tube tulip dome, you should be able to use the telephone as you normally would by holding it up to your ear canal opening. Finding the optimal position for holding a telephone whilst using a hearing instrument may require practice, and the following suggestions may be helpful if you are experiencing difficulties hearing whilst using the telephone.

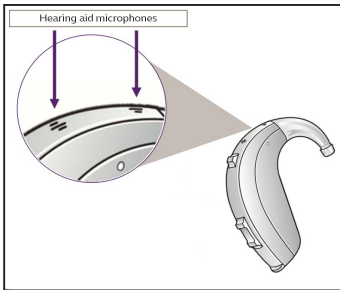

- 1 Find the hearing aid microphones

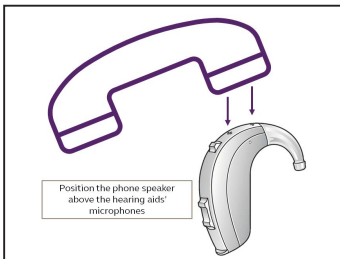

- 2 In order to hear better on the telephone the speaker is best positioned above the hearing aid microphones

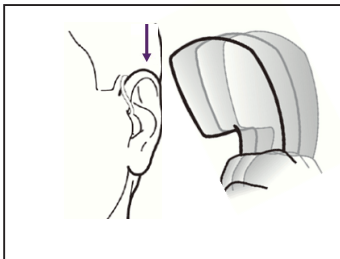

- 3 Place the telephone speaker above the hearing aid microphones rather than directly to your ear