
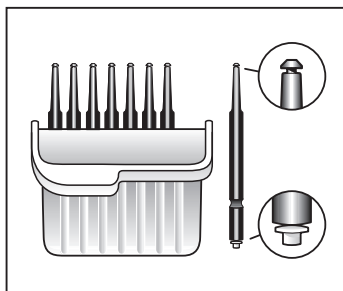


# How to change a wax filter in a Danalogic custom hearing aid

The wax filter in your hearing aid is located at the end of the hearing aid which inserts into your ear.

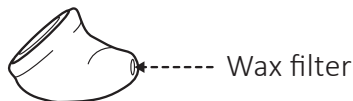

The wax guard tool has two functions:

- A removal tip to collect the used wax guard
- A replacement tip with a white wax guard



1. Insert the removal tip of the tool into the used wax guard firmly.



2. Slowly pull the wax guard straight out.

## Changing wax filter (continued)



3. Turn the tool around and insert the replacement tip of the tool into the hole where you removed the old wax guard.



4. Gently press the replacement tip straight into the hole until it lies flush with end of the hearing aid.



5. Pull the tool straight out. The replacement wax guard is now inserted into the hearing aid.



For further support, scan the QR code or visit our patient support portal at: **[danalogic.co.uk/patient-portal](https://danalogic.co.uk/patient-portal)**