



How to clean your earmould

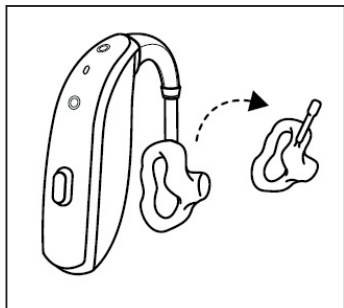

First of all, remove the earmould and tubing from your hearing aids.

Next, clean the earmould using warm soapy water. Then rinse the mould to remove any soap and shake out any water that has collected in the tubing.

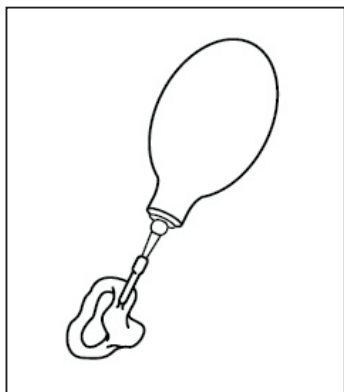

Before reattaching the earmould to the hearing aid, ensure it is fully dry and remove any residual water from the tube by either shaking firmly or using a puffer.

If after reassembling your hearing aid you cannot hear with it this is likely to be due to some water remaining in the tubing. If this is the case, remove the earmould and tubing as you did when washing it and give the earmould and tubing a firm shake to remove any remaining droplets of water. Reassemble the hearing aid and earmould once you have done this.



For further support, scan the QR code or visit our patient support portal at: danalogic.co.uk/patient-portal