




How to change the domes on your hearing aids

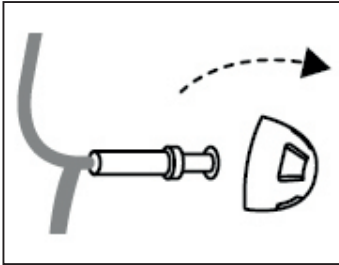

Remove the used dome by pulling it off the end of the slim tube or receiver.

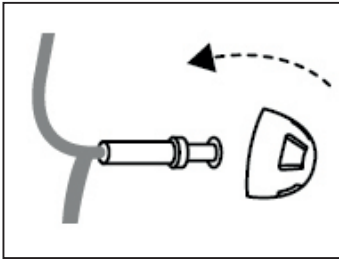


Push the new dome onto the slim tube or receiver firmly.

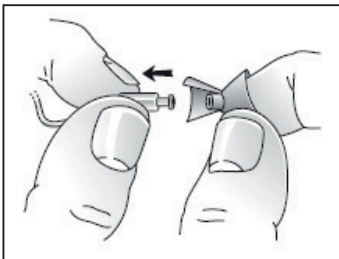

With a tulip dome you may want to pull the largest petal forwards to open the tulip dome and make it easier to push on firmly.

How to change the domes on your hearing aids (continued)

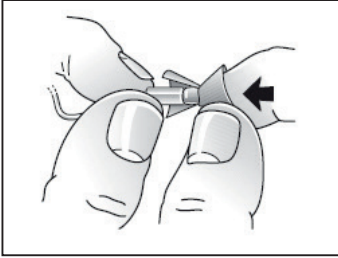

Check that the dome is securely in place by carefully lifting the lower part of the dome and verifying that the dome completely covers the end of the slim tube or receiver.



For further support, scan the QR code or visit our patient support portal at: danalogic.co.uk/patient-portal