



# Cleaning and replacing slim tubes for Ambio and Ambio Smart

## Cleaning

1. Remove the slim tube from your hearing aid by unscrewing it anticlockwise.
2. Wipe down the slim tube and dome with a damp cloth.
3. Now, push the cleaning wire through the top of the slim tube where it connects to the hearing aid. Thread the cleaning wire completely through to fully remove any debris.

### TOP TIP:

*Slim tube and domes should be replaced when the components become stiff, brittle or kinked.*

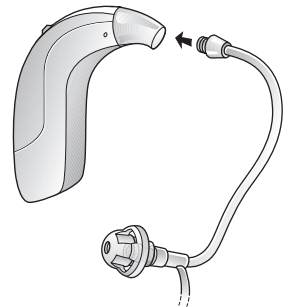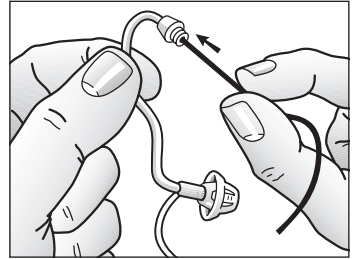

## Replacing

1. Remove the slim tube from your hearing aid by unscrewing it anticlockwise.
2. Attach the replacement slim tube by screwing clockwise.



For further support, scan the QR code or visit our patient support portal at: [danalogic.co.uk/patient-portal](https://danalogic.co.uk/patient-portal)