



Inserting and removing slim tubes


1. Start by hanging the hearing aid over the top of your ear.

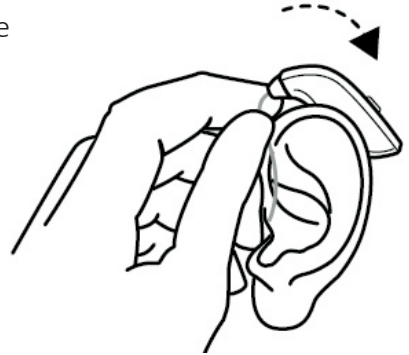

2. Next, hold the tube where it bends and gently push the dome into your ear canal.

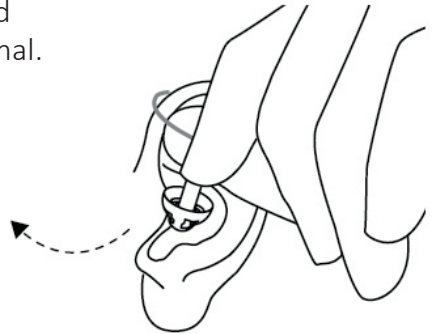


3. Push the dome far enough into your ear canal so that the tube rests flush against your head.

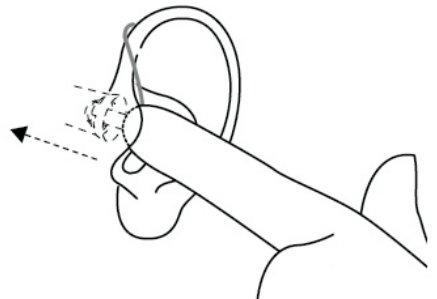


4. Finally, there is a retention cable on each tube which should be placed into the bowl of your ear.

Inserting and removing slim tubes (continued)

5. To remove the hearing aid, first lift the hearing aid off your ear.

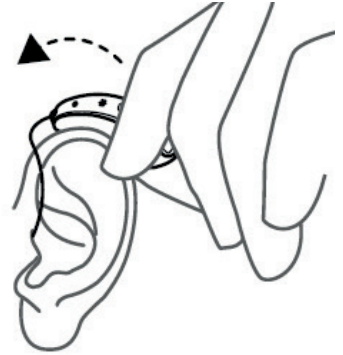

6. Next, hold the tube where it bends and gently pull it out of your ear canal.

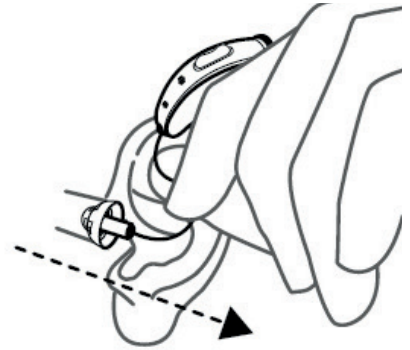

For further support, scan the QR code or visit our patient support portal at: danalogic.co.uk/patient-portal