




How to retube your earmould

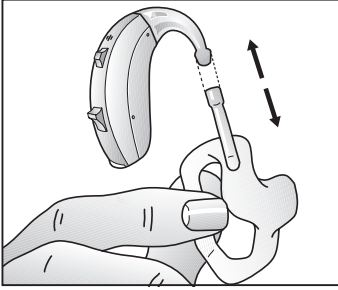


1. Remove your earmould from your hearing aid and then pull the tubing from the earmould.

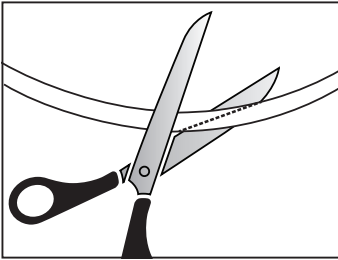

2. Cut approximately the last 5cm of pre-bent piece of tubing to a tapered end.

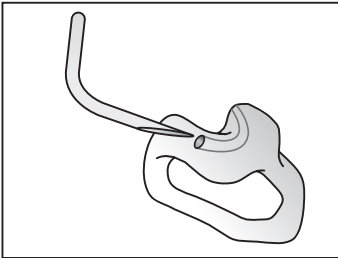


3. Feed the pointed end through the hole in the earmould from the outside as shown, until it emerges at the other end. If you have two holes in your earmould push the tubing through the bigger hole at the top.

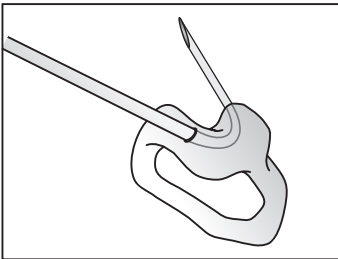


4. Pull the pointed end of the tubing through the earmould until the other end of the tubing is vertical and sits flat against the mould.


How to retube your earmould (continued)

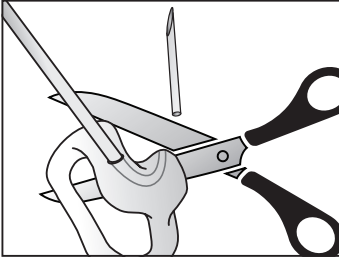

5. Cut off the pointed end as close to the earmould as possible. Ensure there are no rough edges of the tube left sticking out as this can cause irritation in your ear.

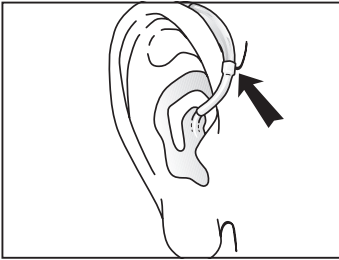

6. Now you need to adjust the length of the tubing to fit your ear. Place the old piece of tubing, (which you removed earlier), alongside the newly fitted tube, measure and cut. Or place the hearing device over your ear, and mark the point where it crosses your hearing aid as shown and cut.

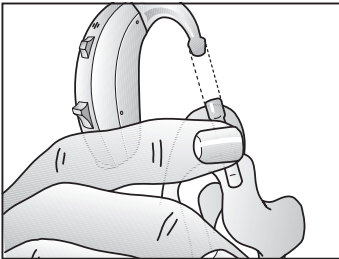

7. To reattach your hearing aid to the earmould, squeeze the top of the tube and push the earmould and hearing aid together.

TOP TIP: *Earmould tubing may become stiff, brittle, or discoloured over time. It is usually recommended that this is changed at least once every 6 months. If you have any issues when changing your earmould tubing please contact your local Audiology department.*



For further support, scan the QR code or visit our patient support portal at: danalogic.co.uk/patient-portal