


How to turn your hearing aids on and off, and change the battery

Turning On

With a battery inserted, close the battery door completely to switch your hearing aids on.

Turning Off

Open the battery door until a click is felt to turn your hearing aids off.

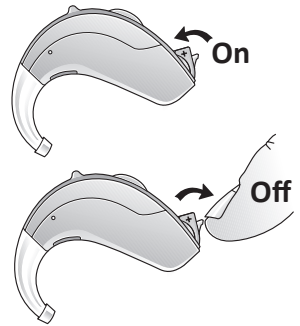

Changing batteries

When the battery power is low

- Your hearing aids will beep when the battery is starting to run out.
- This warning beep will repeat every 15 minutes until the battery has completely run out.

How to turn your hearing aids on and off, and change the battery (continued)

To change the battery

- Remove the sticky label from the new battery.
- Open the battery door.
- Remove the used battery by pulling it upwards.
- Insert the new battery in its place. Make sure that the flat side (marked +) is facing upwards.



TOP TIP:

To save the battery, make sure you turn your hearing device off when you are not using it.



For further support, scan the QR code or visit our patient support portal at: danalogic.co.uk/patient-portal