


# How to turn on and off your rechargeable hearing aids

To turn your rechargeable hearing aids on, press the button for 5 seconds and you will see a 2-second green blink from the LED light.

To turn your rechargeable hearing aids off, press the button for 5 seconds and you will see three 1-second green blinks from the LED light.

If your desktop charger is plugged into a power supply and turned on, inserting the hearing aids into the charger will automatically turn them off. Conversely, removing them from the charger will turn the hearing aids on.

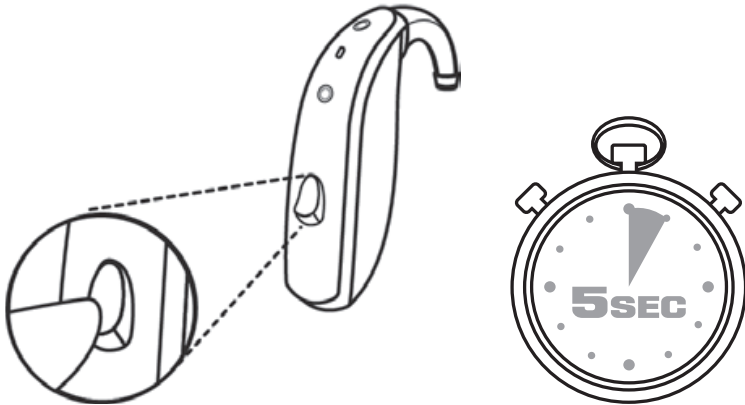

For further support, scan the QR code or visit our patient support portal at: [danalogic.co.uk/patient-portal](https://danalogic.co.uk/patient-portal)