

Battery locks

Instructions for the Ambio & Ambio Smart 62/77/88 models.

To lock the battery door:

Ensure the battery door is firmly closed. Insert the battery lock tool into the slot shown here on the right and push the slider to the right.



To unlock the battery door:


Insert the battery lock tool into the slot shown here on the right and push the slider to the left.



Instructions for the Ambio & Ambio Smart 98 models.


To lock the battery door:

Insert the battery lock tool into the slot shown here on the right and push the slider to the left. A white dot will appear.

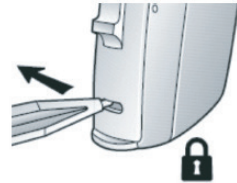

To unlock the battery door:

Insert the battery lock tool into the slot shown here on the right and push the slider to the right. The white dot will disappear.

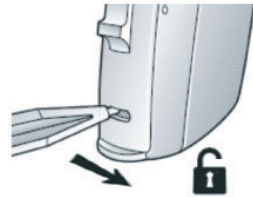

For further support, scan the QR code or visit our patient support portal at: danalogic.co.uk/patient-portal