


The programme button

- Your hearing aids can have up to 4 different programmes. The programmes can be selected using the button on the back of the hearing aids. An example of where you will find the button can be seen on the image here.
- To change programmes, press the button. The number of beeps you hear indicates the programme you are in.
1 Beep = Programme 1
2 Beeps = Programme 2, etc.
- The programmes will cycle through in a numerical order. For example, programme 2 will follow after programme 1.
- When you switch your hearing aids off and on again, they will always return to programme 1.

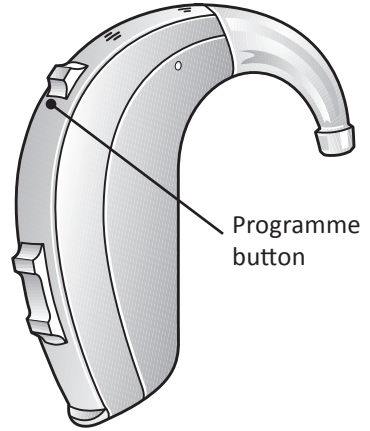

Record your programmes here:

	Programme Name	When to use
Programme 1		
Programme 2		
Programme 3		
Programme 4		



For further support, scan the QR code or visit our patient support portal at: danalogic.co.uk/patient-portal