



How to position the telephone


If your hearing aids are fitted with a slim tube and an open/tulip dome, you should be able to use the telephone as you normally would by holding it up to your ear canal entrance. Finding the optimal position for holding a telephone whilst using your hearing aids may require practice, and the following suggestions may be helpful if you are experiencing difficulties hearing whilst using the telephone.

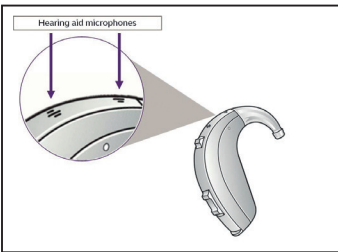

1. Find the hearing aid microphones.

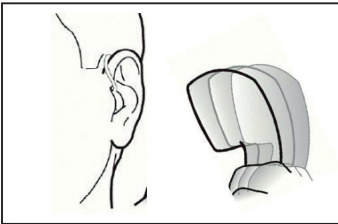

2. In order to hear better on the telephone, the speaker is best positioned above the hearing aid microphones.

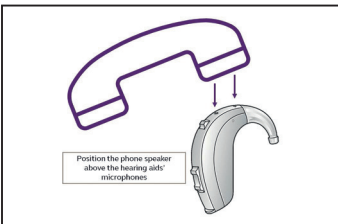

3. Place the telephone speaker above the hearing aid microphones rather than directly to your ear.



For further support, scan the QR code or visit our patient support portal at: danalogic.co.uk/patient-portal