


Volume control – toggle

The volume control will allow the volume of your hearing aids to be increased or decreased.

1. To increase the volume, press the upper end of the toggle.
2. To decrease the volume, press the bottom end of the toggle.

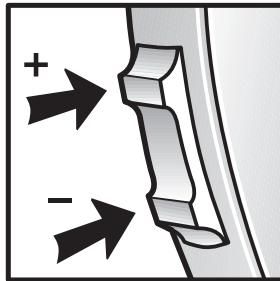

When you increase or decrease the volume you will hear a beep for each press. When you reach the upper or lower limit of the volume range you will hear a double beep. These volume changes are temporary and when you restart your hearing aid it will return to the volume set by your Audiologist. Please use the BeMore app if you would like to save these volume control settings by using the 'Favourite' feature.



For further support, scan the QR code or visit our patient support portal at: danalogic.co.uk/patient-portal