


Danalogic hearing aids - How to use your Desktop Charger

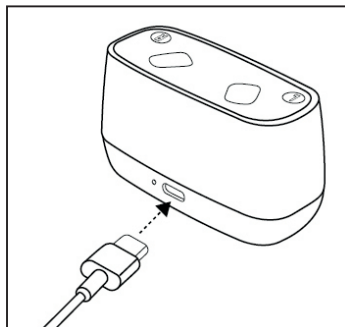


1. Ensure that the charger is plugged into the mains and the power socket is switched on when you are ready to charge your hearing aids.



2. Place your hearing aids into the charging bays as shown. The left hearing aid goes into the 'L' bay and the right into the 'R' bay. The slim tubes/earmoulds of your aids will be facing away from you towards the power cable. Once the aids are fully inserted, the LED lights on them will light up to show they are charging.

Danalogic hearing aids - How to use your Desktop Charger (continued)

- When the hearing aids are charging the LED lights will blink slowly. Once the aids are fully charged the LED lights will remain on without blinking until your hearing aids are removed from the charger or the power to the charger is turned off.

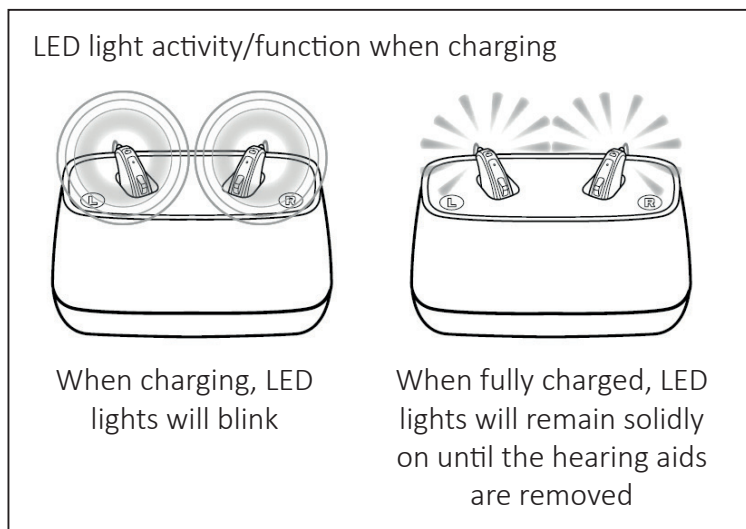

- There is up to 23 hours of use from a full charge and up to 16 hours if you are streaming for 50% of the time. A full charge of the battery takes up to 3 hours.



For further support, scan the QR code or visit our patient support portal at: danalogic.co.uk/patient-portal