


The programme button

- Your hearing device can have up to 4 different programmes. The programmes can be selected using the button at the top, on the back of your hearing device.
- To change programmes press the button. The number of beeps you hear indicates the programme you are in.
1 Beep = Programme 1
2 Beeps = Programme 2, etc.
- The programmes cycle through 1 to 2 to 3 to 4, then back to 1.
- When you switch the hearing device off and back on again it will always return to programme 1.

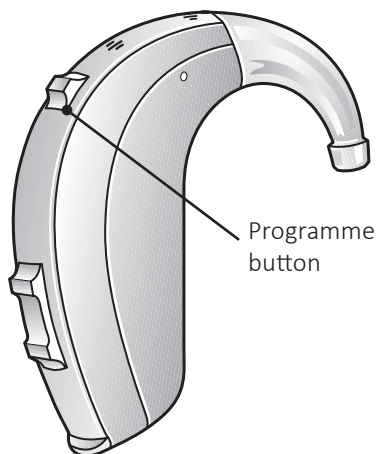

Record your programmes here

	Programme Name	When to use
Programme 1		
Programme 2		
Programme 3		
Programme 4		

Your hearing device has been set up to beep when you change between programmes.

Yes

No

Audiologist contact details: