

# Volume control - wheel

The volume control is situated on the back of your hearing device above the battery drawer and below the programme button. The volume control is a wheel and is marked 1 to 4.

**1 is the quietest and 4 is the loudest.**




is about the right volume for you.

Although you may find it useful to adjust the volume in different listening situations.

- **TO REDUCE** the volume  
– roll the wheel **downwards**
- **TO INCREASE** the volume  
– roll the wheel **upwards**

The volume wheel will stop when you reach the upper and lower limits of the volume range.

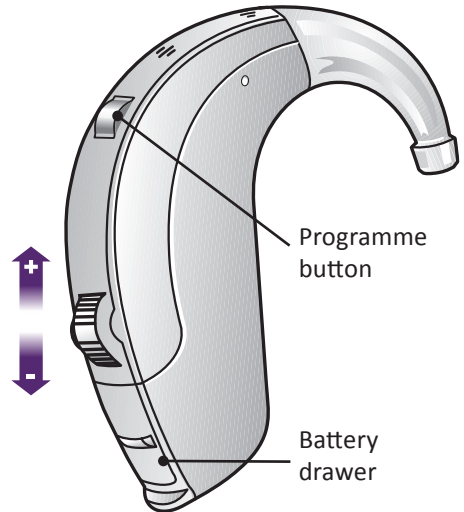

Audiologist contact details: