


General cleaning and maintenance advice

Here are some handy tips to keep your Danalogic Hearing aids in the best possible condition:

- Keep your hearing aids clean and dry. You can use a soft cloth or tissue to remove grease or moisture and ensure your hands are clean and dry
- Do not use water or solvents to clean your hearing aids, or immerse them in water or other liquids
- Avoid dropping your hearing aids on hard surfaces
- Do not leave your hearing aids in or near direct sunlight, such as in a hot, parked car, as excessive heat can cause damage to the casing
- Do not wear them while showering, swimming, in heavy rain or in a moist atmosphere such as a sauna
- Remove your hearing aids when applying cosmetics, perfume, aftershave, hairspray or lotion as these can damage your hearing aids
- In addition to these, please check your thin tube, dome, tubing, earmould for any obvious blockages and clean appropriately

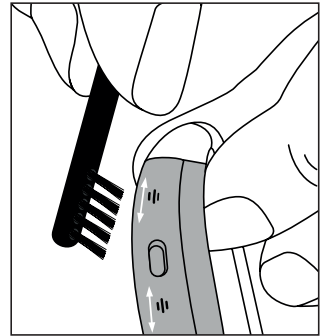

On the top / spine of the hearing aid are two microphone inlets. (On either side of the top push button). The microphones are protected by a sound-transparent filter; but, like any filter, can get clogged with dust and debris.

Use a hearing aid cleaning brush to brush over each microphone inlet every other day.



For further support, scan the QR code or visit our patient support portal at: danalogic.co.uk/patient-portal